


RECOVERYRX GENERAL SURGERY KIT 5CM | 8CM | 12CM

STEP 1: 5cm & 8cm: First pull the white paper tab and then the clear plastic tab from the RecoveryRx to activate. A red light confirms that RecoveryRx is on and working. Squeeze the rim of the foam together to reseal the opening.

STEP 1: 12cm: Hold the white foam portion of the RecoveryRx loop and gently pull and remove the clear tab. Remove the white plastic tab DO NOT discard the tab, it can be reinserted to turn off the RecoveryRx loop when not in use. A faint red light confirms the RecoveryRx device is activated.


5cm

The electronics and battery are in this module.

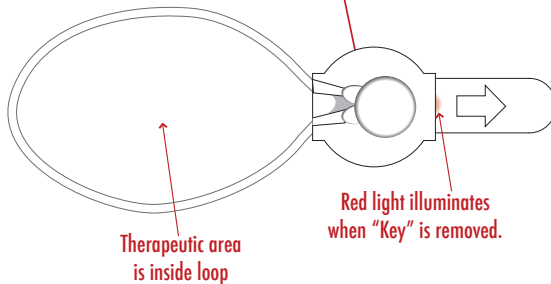


8cm

The electronics and battery are in this module.

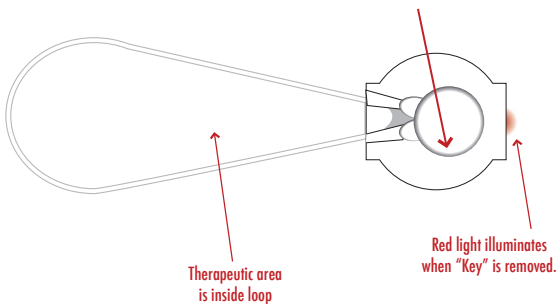

12cm

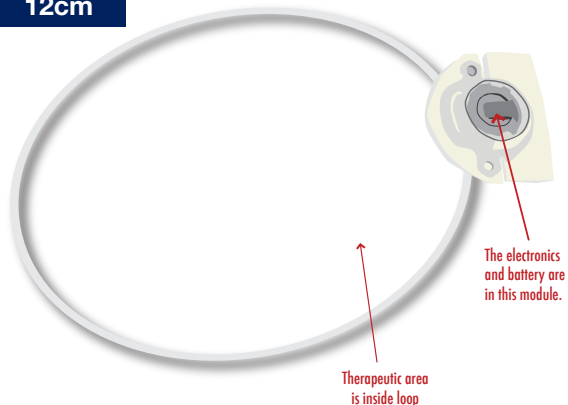

5CM LOOP DEVICE EYE SURGERY

Gently bend the loop of the device so it goes around the eye as illustrated below. The loop should be positioned as close as possible to the skin around the eye. It is not necessary for the loop to touch the skin. Use medical grade tape to secure the foam portion of the device to the side of the face.



8CM LOOP DEVICE JAW SURGERY

The Loop on the RecoveryRx is the treatment area. Position the smooth flat side of the RecoveryRx on the face so that the area of treatment is inside the loop. Use medical grade tape to secure the RecoveryRx loop in place.




8CM LOOP DEVICE NOSE SURGERY

Apply tape to the nose using standard method.

Adapt the wire 'loop' to the outline of the patient's nose. Loop should be flush with skin surface; on/off tab should be located on the glabella.

Place splint on top and secure. The loop itself should 'fit' around the outside recessed edge of the splint. Avoid trapping the loop under the splint in order to ensure the loop does not imprint on skin.

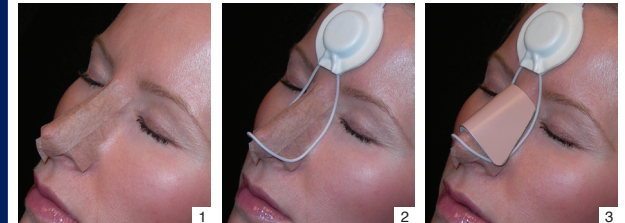

12CM LOOP DEVICE C-SECTION

First pull the white paper tab and then the clear plastic tab from the RecoveryRx to activate. A red light confirms that RecoveryRx is on and working. Squeeze the rim of the foam together to reseal the opening.

The Loop on the RecoveryRx is the treatment area. Position the smooth flat side of the RecoveryRx directly over the area after C-section surgery.




12CM LOOP DEVICE BREAST SURGERY

Place RecoveryRx over each breast in the operating room to be worn continuously for 48 hours to reduce pain and swelling.

Do not bind or wrap an injured muscle. Early moving and stretching of an injured muscle is key to recovery.


The night of surgery, start the patient on pain medication as well as a range of motion arm exercises.

Send them out the next day to tell their friends and family about your care.

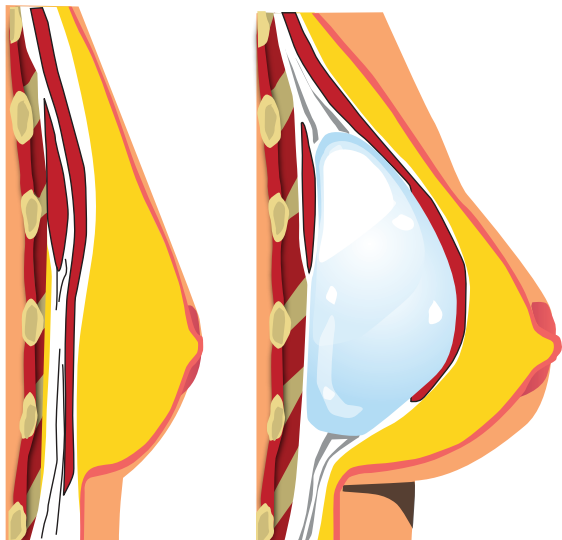

Before
Surgery

Subpectoral
Implant

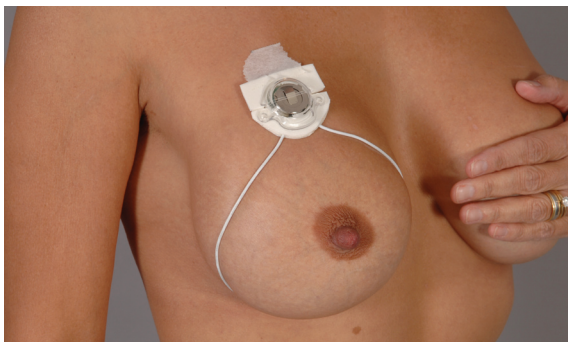

FREQUENTLY ASKED QUESTIONS

What is RecoveryRx? RecoveryRx is a miniaturized medical device that delivers continuous electromagnetic therapy to restore damaged cells. Applied post-surgically or to an injury, RecoveryRx quickly relieves edema, inflammation and pain, providing patient comfort while reducing bruising and speeding up the onset of natural healing.

How does RecoveryRx work? Tissue cells are tiny, electrically charged machines. When traumatized by surgery or injury cell's electrical charge is diminished, which causes normal cell functions and operations to shut down. Chemical signals that cause inflammation are released, resulting in pain and inhibiting the cell communication necessary for healing to begin. RecoveryRx induces a mild electrical current into damaged cells at frequency which recharges the cell, stops the release of pain and inflammatory mediators, increases blood flow and re-establishes normal cell interaction. As a result, your body can begin to heal faster and you experience less swelling, pain and bruising.

Is RecoveryRx Safe? RecoveryRx is extremely safe and comfortable to wear. The low level current that is gently pulsed into damaged tissue is not discernible, even though the device is continuously working to help you heal.



WARNINGS

RecoveryRx should not be used on persons with a cardiac pacemaker, cardioverter defibrillator, neurostimulator or any active medical or metallic implant that is within the area of application.

RecoveryRx should not be used by women that are pregnant or think they may be pregnant.

RecoveryRx should not be used by skeletally immature persons.

RecoveryRx is not a sterile device. If using over an open wound, place the RecoveryRx over sterile gauze.

Discontinue use if rash or skin irritation results. Consult a qualified health care practitioner before using RecoveryRx again.

Do not stand, walk, or put body weight on RecoveryRx as this may damage the circuitry.

RecoveryRx can be disposed of in a normal trash container.

RecoveryRx will not interfere with other electronic equipment in use.

Always consult a qualified health care practitioner to evaluate your condition or injury prior to using RecoveryRx. If pain or swelling persist or worsen, discontinue use and seek medical attention immediately.



U.S. & Foreign Patents Pending

The product expiration date is on the box label.
Look for the symbol  for date.

Safe for Diabetics, Arthritics, Elderly,
the Bed Ridden and on Open Wounds

1-866-757-2284
Manufactured by:
BioElectronics Corporation
4539 Metropolitan Court
Frederick, MD 21704
Stock symbol: BIEL
www.BioElectronicsCorp.com



Read instructions before using

DFU-RXGEN-ITAL-0001